


The syringe in your package may have either a syringe tip or a closed cap.
The preparation instructions below are similar for both options.

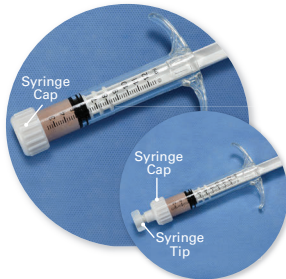

STEP 1

Open the pouch and remove the syringe. Confirm the syringe cap and syringe tip (if applicable) are securely locked.




STEP 2

Place the syringe in the ambient sterile bath for 10-15 minutes, ensuring the contents are submerged.




STEP 3

Remove the syringe from the sterile bath and unscrew the syringe cap from the top as shown above.

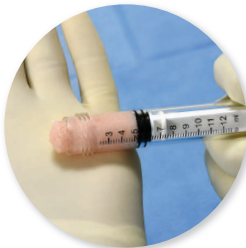

STEP 4

Apply slight pressure to the plunger and extrude product out of the syringe into the palm of your hand.

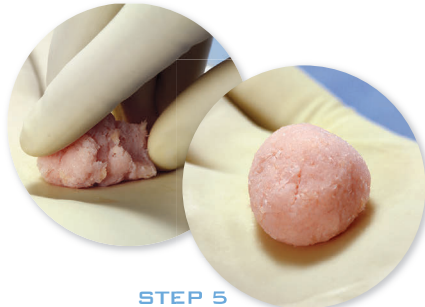

STEP 5

Curigraft is now ready to mold to desired shape. Product remains viable for up to 4 hours on the back table in an operative field.

Curiteva has used reasonable efforts to provide accurate information herein, but this information should not be construed as providing clinical advice, dictating reimbursement policy, or as a substitute for the judgment of a health care provider.